



Step 2: *BAP1-TPDS (BAP1 Tumor Predisposition Syndrome) Genetic Test Sample Requirements*

U.S. insurance patients: a completed *Form 1c: U.S. Insurance Information* must be provided.

All submitted samples must be labeled with at least two patient identifiers (ie. name and date of birth)

Sample Requirements

Blood sample:

- 10 mls in lavender-topped EDTA tubes at room temperature, to be received within 5 days after draw.

Sample Identification:

- Label each sample with **at least two** patient identifiers (e.g. named and date of birth).

Step 3: *Shipping Requirements*

Multiple separated samples may be shipped in one package. Place multiple biohazard bags containing labeled samples into one box. Multiple boxes can be shipped in one courier envelope.

Shipping Instructions

- Ship samples to Impact Genetics at address shown on this page using a courier envelope.
- Include Informed Consent and Requisition Forms (**1a** and **1b**) with the samples. Patients in the U.S. must also include U.S. Insurance Information (**Form 1c**) if required and not provided previously.
- Complete appropriate Air Waybill. If you cannot use FedEx or Purolator, please contact us.
- Place Air Waybill in the document pouch.
- For samples from outside of Canada, complete and sign appropriate customs forms (provided and available on our website; phone us if help is required). Place the customs forms in the document pouch.
- Within Canada, use **Purolator Express** (next-day) or **FedEx Priority** service (next-day). Outside of Canada use **FedEx Priority** service (next-day) and **use a FedEx "Clinical Pak"**. If you cannot use Purolator or FedEx, please contact us.
- Provide us with the parcel tracking number soon after courier pick-up: **647-478-4902, info@impactgenetics.com**.
- For emailed PDF FedEx waybills and customs forms, please contact us directly.

Send to Impact Genetics

mail: Impact Genetics
1100 Bennett Road - Unit 4
Bowmanville, ON L1C 3K5

tel: 1-877-624-9769

fax: 905-697-9786